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# Vaginal applicator

**Approved indications:** Used for insertion of capsules into the vagina.


---

## Instructions for use:

---


**1** Important: Before insertion, wash your hands with water and soap.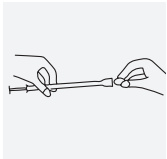

**2** At the end of the applicator there is a plunger; pull it out and insert the soft capsule into the other end of the applicator, so that the capsule is slightly protruding outwards (do not worry - the capsule cannot go wholly into the applicator).

**3** Lie on your back, with your knees bent and pointing upwards.

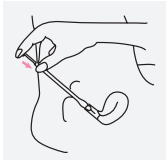**4** Using the applicator, insert the capsule gently into the vagina, as deep as possible, without applying force or causing any discomfort. The capsule will be released when you press down on the plunger.

**5** Leave the plunger pressed and remove the empty applicator.

**6** Wash the applicator and your hands with soap and warm water.

**7** Repeat the process to insert an additional capsule, as necessary.

---

**In which cases the device should not be used:** Use according to the doctor's instructions.

**Precautions:** Keep away from children | Do not flush the applicator down the toilet.

**Storage:** Protect from light.

---

**Name and address of the manufacturer:** Hubert De Backer n.v. Laagstraat 59, Temse, Belgium

**License holder and the address:** CTS Ltd. 4 Haharash St., Hod Hasharon, 1-700-500-220

Registration number in the Medical Device Registry: 350431